





RelyX™ Cements – Product Family Chart



PRODUCT BRAND	PRODUCT NAME	INDICATIONS	CEMENT TYPE	KEY ADVANTAGES
RelyX™	Unicem Self-Adhesive Universal Resin Cement 	Universal cement product for metal and all-ceramic/composite restorations.	Self-adhesive resin cement	<ul style="list-style-type: none"> • Saves time by eliminating steps • Lowers potential for patient sensitivity • Offers a convenient, unidose capsule • Formulated to be self-adherent and moisture tolerant
RelyX™	Luting Plus Cement 	For routine crown and bridge cementation. For PFM, all-zirconia or all-alumina strengthened core ceramic restorations (such as Lava™ or Procera®), metal (includes stainless steel crowns), inlays and onlays, endodontic pins and posts and orthodontic appliances.	Resin-modified glass ionomer	<ul style="list-style-type: none"> • Convenient paste-paste delivery in the Clicker™ Dispenser • Consistent mix ratios • Enhanced formula for better performance • Easy clean-up • Low post-operative sensitivity • Sustained fluoride release
RelyX™	Veneer Cement 	For veneer cementation procedures.	Resin cement	<ul style="list-style-type: none"> • Customer-preferred delivery and shading system • Try-in pastes offer excellent color match to final cured cement • Light-cure-only system provides excellent color stability • Easy to clean up • Easy to dispense, apply and seat • Cement cures with a wide range of light sources
RelyX™	Temp NE Cement Temp E Cement 	For temporary or provisional cementation procedures.	Zinc Oxide Temporary Cement • Non-eugenol • Eugenol	<ul style="list-style-type: none"> • Strong adhesion • Quick and easy excess removal • Optimal fit • Easy provisional removal • Available in both eugenol and non-eugenol formulas
RelyX™	Luting Cement 	For routine crown and bridge cementation. For PFM, metal, and strengthened core ceramics.	Resin-modified glass ionomer	<ul style="list-style-type: none"> • Easy to mix, load and seat due to its mousse-like consistency • Easy clean-up • Zero solubility for enhanced marginal integrity¹ • High fracture toughness • Sustained fluoride release • Virtually no post-operative sensitivity²
RelyX™	ARC Adhesive Resin Cement 	For indirect porcelain, ceramic, and composite restorations.	Resin cement	<ul style="list-style-type: none"> • Recommended for indirect restorations • Easy to use • Dispenses uniform, procedure-specific dosages • Provides easy excess removal

¹3M ESPE laboratory testing according to ISO 9917: 1991 (E) acid erosion test.
²JADA, Vol. 126, June 1995, pp. 781–782, Author: Dr. Gordon Christensen.

RelyX™ Cement Products—Product Usage by Indication



RESTORATION TYPE	RelyX™ Unicem Self-Adhesive Universal Resin Cement	RelyX™ Luting Plus Cement	RelyX™ Veneer Cement	RelyX™ Temporary Cements	RelyX™ Luting Cement	RelyX™ ARC Adhesive Resin Cement
Metal/PFM				For luting of indirect temporary restorations (inlays, onlays, crowns and bridges).		
- Inlays/Onlays	●	●			●	■
- Crowns	●	●			●	■
- Bridges	●	●			●	■
- Endodontic Posts	●	●			●	●
- Maryland Bridge	NI	NI			NI	●
- Orthodontic Appliances	NI	●			●	NI
Porcelain/Ceramic¹						
- Crowns	●	NI			NI	●
- Bridges	●	NI			NI	●
- Inlays/Onlays	●	NI			NI	●
- Veneers	NI	NI	●		NI	NI
All-zirconia or All-alumina Strengthened Core Ceramic i.e., Lava™ or Procera®						
- Crowns	●	●			●	●
- Bridges	●	●			●	●
Composite/Reinforced Polymer²						
- Crowns	●	NI			NI	●
- Inlays/Onlays	●	NI			NI	●

- Recommended product. Designed specifically for stated application.
- Secondary recommendation where the preparation has left minimal tooth structure and added retention is required.
- NI Not recommended or indicated.

¹ Porcelain/Ceramic not having all-alumina cores. These include conventional feldspathic porcelains, pressable and castable ceramics. Examples of these types are Vitadur Alpha, Finesse™ Ceramco II, IPS Empress™ IPS Empress™ 2, In-Ceram Spinell, OPC, etc.

² Composite/Reinforced Polymer. Examples of these types are belleGlass™ HP, Targis™/Vectris™ Artglass™ Concept, Sculpture/FibreKore, etc.




Dental Products

3M Center
Building 275-2SE-03
St. Paul, MN 55144-1000
USA

3M Canada

Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-800-265-1840 ext. 6229

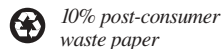

Printed in USA
© 3M ESPE 2004

3M, ESPE, RelyX and Lava are trademarks of 3M or 3M ESPE AG. Procera is a trademark of Nobel Biocare. Finesse is a trademark of Dentsply Ceramco. Empress, Targis and Vectris are trademarks of Ivoclar Vivadent, Inc. belleGlass is a trademark of KerrLab. Artglass is a trademark of Heraeus Kulzer, Inc.